Malleo PROMASTER

D: Sprunggelenkbandage mit verstärktem propriozeptiven Effekt
GB/USA: Ankle brace with reinforced proprioceptive effect
F: Chevillère à effet proprioceptif renforcé
I: Cavigliera con effetto proprioceptivo rinforzato
E: Tobillera con efecto proprioceptivo reforzado
P: Artelheira com efeito proprioceptivo reforçado
NL: Enkelverband met versterkt proprioceptief effect
DK: Ankelbind med forstærket proprioceptiv virkning
S: Ankelledsförband med proprioceptiv verkan
FIN: Tukeva proprioseptiivinen nilkkatuki
GR: Επιστραγάλιδα με ενισχυμένη ιδιοδεκτικότητα
CZ: Kotníková ortéza se zesíleným účinkem propriocepce
PL: Orteza skokowo-stopowa ograniczająca supinację stopy
EST: Tugevdatud propriotseptiivse toimega hüppeliigesetugi
LV: Pēdas locītavas ortoze ar stiprāku ietekmi propriocepcijai
LT: Čiurnos įtvaras su sustiprintu propriocepčinu poveikiu
SK: Členková ortéza so zosilneným proprioceptívnym účinkom
SLO: Gleženjski povoj z okrepljenim proprioceptivnim učinkom
H: Megerősített proprioceptív boka ortézis
RO: Glezneră cu efect proprioceptiv pronunțat
BG: Ортеза за глезен с подсилено
проприоцептивно действие

THUASNE

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Ankle brace with reinforced proprioceptive effect

Properties:

- Brace made of elastic knit
- 3D knit at the malleoli
- Malleolar pads
- Latex-free

Indications:

Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease; Moderate joint instability

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the two malleoli are positioned in the 3D knit zones specially designed for this purpose. The ankle brace can be worn against the skin or on top of a well fitting sock.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions:

- See product label
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

Composition:

See product label

Explanation of the reinforced proprioceptive effect:

Orthoses in the PROMASTER range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- I** THUASNE ITALIA Srl. · Via Luigi Capuana, 27 · 20825 Barlassina (MB)
- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE BEGAT AB · Box 92050 · 120 06 Stockholm
THUASNE DEUTSCHLAND GmbH