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### Savybės

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## **EN** Clavicular immobilization straps

### Properties

- Two straps crossed over at the back and fastened to an anchoring pad.
- Single size, product to be cut to the patient's size.
- Two Velcro parts for fastening the orthosis after the straps have been cut.
- Comfortable pads on the crossed straps.

### Indications

- Treatment of collar bone fractures, with or without displacement.

### Contraindication

Do not apply the product in direct contact with broken skin.

### Preparation

- Measure the shoulder width and identify the appropriate size, referring to the size chart (sizes 1 to 4).
- Cut the straps to the identified size, following the cutting marks.
- Fasten the product using the Velcro parts.

### Fitting instructions

- The help of a third party is required to fit this product.
- 1. After first having cut the straps to size, place the Velcro parts at the end of the straps.
- 2. Position the straps around the collar bones and under the armpits.
- 3. Pass the straps through the buckles.
- 4. Adjust the compression and fasten the straps with the Velcro parts.
- 5. Adjusting the orthosis every day will ensure better reduction of the fracture.

### Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs.

Store at room temperature, preferably in its original packaging.

For hygiene and performance reasons, do not re-use the product for another patient.

### Washing instructions

- See product label.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Squeeze out excess water.
- Dry away from heat sources.

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