I Elastocalx

Tallonette in viscoelastico

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GB USA Elastocalx

Visco-elastic heel cushions

The Elastocalx heel cushions are manufactured using highly elastic silicone and provide optimum shock absorption for ankle and knee joints, hip and spinal column. Elastocalx alleviates stress on the joints, ligaments, muscles and tendons. The different models each have specific modes of action.

This product requires qualified and individual adaptation to the patient by specialist staff.

Indications/Specific modes of action:

Elastocalx N
Osteoarthritis of the ankle, knee and hip joints, degenerative spinal diseases.
Relief after ankle, knee and hip joint operations, irritations of the Achilles tendon, myotendopathies (long-term, recurrent pain), strain syndromes, differences in leg length, scoliotic pelvis.

Elastocalx S/P
For the correction of supination (raised inside margin of the foot) or pronation (drop in the inside margin of the foot) in the foot and in cases of valgus or varus deformity of the heel (knock-kneed or bow-legged).

Elastocalx Plus
Relief from heel spur (bony spur formation under the heel) pressure by means of a particularly soft silicone insert in the area of the heel spur.

Elastocalx Flex
Relief from heel spur (bony spur formation under the heel) pressure by means of an open, honeycomb construction covering a large area in the region of the heel spur.

Contraindications/Side effects
Larger skin lesions or diseases in the heel area.
No side effects are as yet known when applied and worn as intended.

Insertion of the heel cushions/Precautions

General and Elastocalx N:
The heel cushion and size required for your symptoms was selected by your doctor/hospital. Place the heel cushion flat inside the shoe and press the back margin of the cushion against the heel of the shoe.

Elastocalx Plus:
The heel cushions are marked on the bottom with R and L for the right and left shoes.

Elastocalx Flex:
Fit left or right.

Elastocalx S/P:
In cases of pronation (e. g. valgus) of the foot, place the thick side of the cushion under the inside of the heel, in cases of supination, place the thick side of the cushion under the outside of the heel.
For bow legs (varus, e.g. medial arthritis in the knee joint), place the thick side of the cushion under the outside of the heel (pronate), for knock knees, place the thick side of the cushion under the inside of the heel (supinate). Please use the second heel cushion in the same way.

Precautions/other advice:
Store at room temperature. Do not bring into contact with open wounds. In cases of skin reactions or other changes (e.g. deterioration in the condition) immediately stop use and consult a doctor. Do not use without a prescription from the doctor. The manufacturer is not liable for changes made to the product or inappropriate use.

Care:
Wash off regularly with lukewarm soapy water (mild detergent). Do not use either aggressive detergents or bleach. Wipe dry with a suitable cloth.

This medical device is intended for the treatment of a single patient. To guarantee its efficacy and for hygiene reasons it is not allowed to pass it on to other patients.

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