THUASNE

Epi-GO

D: Ellenbogengelenkbandage mit propriozeptivem Effekt
GB/USA: Anti-epicondylitis bandage with proprioceptive effect
F: Coudière anti-épicondylite à effet proprioceptif
I: Gomitiera con effetto proprioceittivo
E: Codera con efecto proprioceptivo
P: Órtese para cotovelo com efeito proprioceptivo
NL: Verband voor het ellebooggewricht met proprioceptief effect
DK: Albuebind med proprioceptiv virkning
S: Armbågsledsförband med proprioceptiv verkan
FIN: Kyynärside proprioseptiivisellä vaikutuksella
GR: Επιπλωκωνίδα με ιδιοδεκτικότητα
CZ: Bandáž lokte se proprioceptivním efektem
PL: Orteza stawu łokciowego ze efektem proprioceptywnym
EST: Küünarliigese bandaaž ar uzlabotu proprioceptiivo efektu
LV: Elkoņa locītavas bandāža ar proprioceptiivo efektu
LT: Proprioceptinio poveikio alkūnės įtvaras
SK: Ortéza lakťového kĺbu so proprioceptívnym účinkom
SLO: Opora za komolec z proprioceptivnim učinkom
H: Könyökízület-rögzítő proprioceptív hatással
RO: Bandaj pentru încheietura cotului cu efect proprioceptiv
BG: Ортеза за лакътна става с проприоцептивно действие

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Anti-epicondylitis bandage with proprioceptive effect**Purpose:**

Epi-GO is a compression bandage with pads for treating elbow irritation. The bandage may only be used for elbow treatment.

Properties:

- Brace made of elastic knit
- 3D knit & pad opening at the condyles
- Epicondyle pads
- Latex-free

Indications:

Epicondylitis; Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the epicondyles are placed correctly in the special knit area. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

86 % PA (polyamide), 14 % EL (elasthane)

Explanation of the proprioceptive effect:

Orthoses in the GO range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- I** THUASNE ITALIA Srl. · Via Luigi Capuana, 27 · 20825 Barlassina (MB)
- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE BEGAT AB · Box 92050 · 120 06 Stockholm
THUASNE DEUTSCHLAND GmbH